

Ava™ Recliner Instructions


See next page for important safety information.

Lock /Unlock


All wheels roll, swivel locked.

All wheels roll and swivel.


All wheels locked.

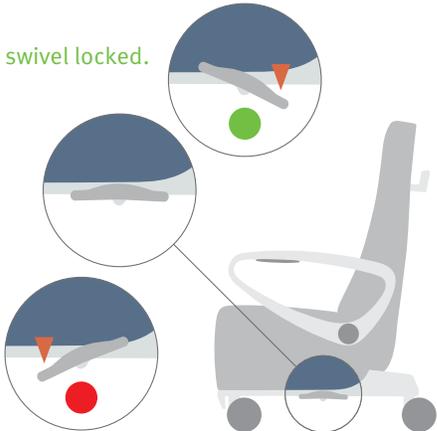


Recline Back

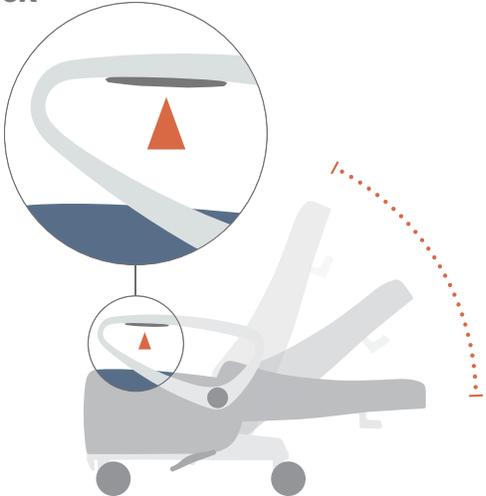

Elevate Footrest

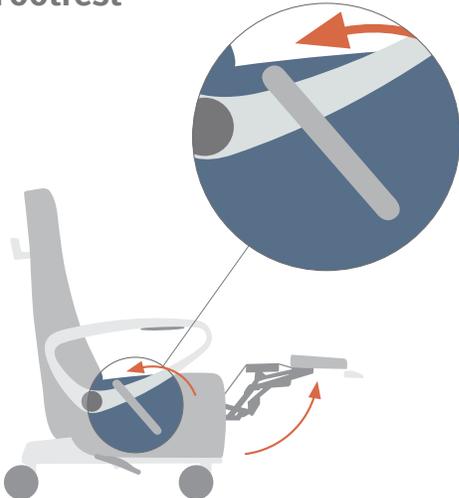

Pivot Arms

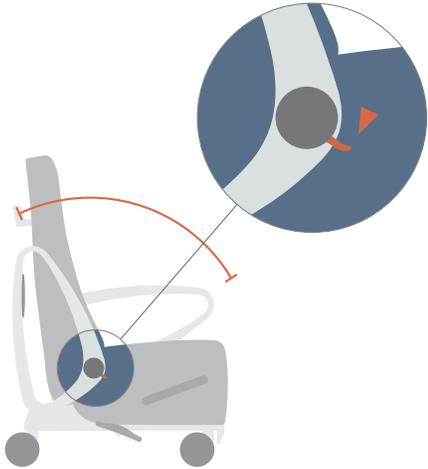

Ava™ Recliner Instructions

Important Information:

Read all of the instructional information on both sides of these tags. If any information is not understood, ask the care provider for assistance before operating the recliner. This product is not a medical device. It is not intended for the diagnosis of disease or use in the cure, mitigation, treatment, or prevention of disease. To avoid personal injury or product damage, inspect and service this chair according to the maintenance instructions. For help or additional information, visit nemschoff.com/ava or call 800 203 8916 in North America.

▲WARNING Failing to follow instructions and warnings could cause an unstable situation resulting in a fall, serious injury or death.

DO NOT use footrest, chair arms or back rest as a seat. Use chair under adult or caregiver supervision.

Casters must be **LOCKED** at all times, except when moving chair. Unlocked casters may allow unintended movement causing injury.

DO NOT move chair by tilting on two wheels.

DO NOT use chair for patient transport.

Keep casters free for debris to ensure proper functioning.

DO NOT put hands, feet, or clothing into the opening between chair seat and back when changing recline position.

STAY CLEAR of reclining mechanism. Failure to do so may result in a pinch hazard and possible injury.

DO NOT exit or enter the chair unless the back is upright and arms are in their down position.

DO NOT use backrest as a seat.

Allow enough room behind chair before reclining.

DO NOT put hands, feet, or clothing into opening between seat and footrest when changing footrest positions.

DO NOT get up from or sit down in the chair unless footrest is completely retracted to prevent the possibility of the footrest from extending unexpectedly.

DO NOT sit on the edge of seat with footrest fully elevated. Failure to do comply may cause the chair to tip resulting in personal injury.

ALWAYS close the footrest when not in use. Failure to comply may result in personal injury.

DO NOT sit on the footrest when fully elevated to avoid a collapse, which may result in a fall.

NEVER allow children to play on or operate chair mechanisms.

CLEAR SPACE around the chair before extending footrest or reclining back. Clear area under footrest when returning footrest to retracted position. Footrest may close unexpectedly causing injury.

DO NOT sit on the chair arm. Failure to comply may cause the arm to break or chair to tip resulting in personal injury.

Care & Maintenance:

With proper care, Nemschoff furniture will provide years of superior performance and satisfaction. Inspect furniture at least once a year to ensure that it is structurally intact and functioning properly. Replace damaged or broken parts, tighten loose fasteners, and lubricate mechanisms in accordance with maintenance protocol.

These instructions are provided for the safety and comfort of anyone using this recliner. **DO NOT** remove this tag. To obtain a replacement copy, or for customer assistance, including care and maintenance information, visit nemschoff.com or call 800 203 8916.

